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APPLICATION INSTRUCTIONS

- Keep the Cleaning Bag out of the reach of children – risk of suffocation!
- Fill the cleaning bag RC-Clean only with microwave durable objects. Do not put any metal pieces or electronic items in the bag.
- Always fill the RC-Clean with 60ml of water. We recommend the use of distilled water.
- Do not use the microwave's grill function or a conventional oven.
- In addition, please observe the instructions for use of your microwave.
- The cleaning bag can be reused up to 20 times. Check off after each use.

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Cleaning Bag for the Microwave

Clean medical products and childcare items easily and quickly.

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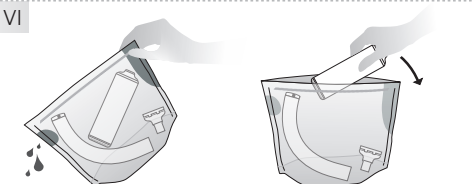
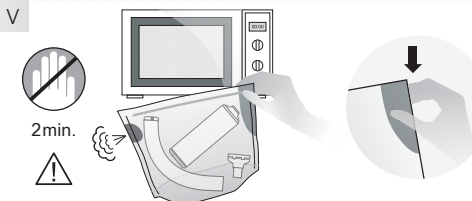
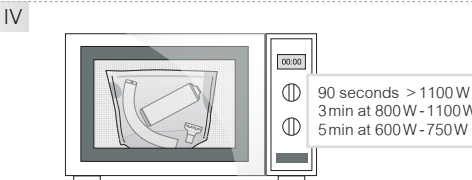
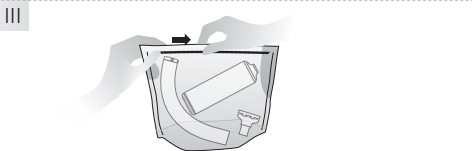
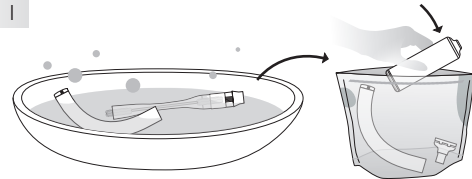
APPLICATION INSTRUCTIONS

I Clean all parts before use with warm soapy water and rinse them with clear water. Lay the products which you wish to clean in the RC-Clean Bag. Do not overfill the bag. Do not put any metal pieces or electronic items in the bag.

II Always fill the RC-Clean with 60 ml of water. We recommend the use of distilled water.

III Seal the cleaning bag well with the pressure lock. The bag must be firmly sealed.

IV Lay the cleaning bag in the middle of the microwave. Adjust your microwave to the correct wattage and duration and start the microwave according to the following table:
 90 seconds at > 1100 W
 3 minutes at 800 W - 1100 W
 5 minutes at 600 W - 750 W

V Open the microwave after the time has expired and let the RC-Clean cool off for a minimum of 2 minutes. Take hold of the bag only by the safety zone and take it out of the microwave. Please note that when taking out or opening the RC-Clean hot steam or water can escape.

VI Pour the excess water carefully out of the cleaning bag through the steam valve. Open the cleaning bag and take the cleaned pieces out. Wash your hands before touching the cleaned pieces. Let all the pieces air dry completely. Do not rinse or rub dry. The cleaned pieces are then ready for use.

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